


blink[®] intensive soothing eye drops 10ml

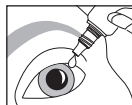

(EN) Long lasting relief from the feeling of dry, tired eyes

An advanced lubricating eye drop specifically formulated to provide soothing comfort for eyes that are dry, irritated, or uncomfortable.

Indications

Irritated, uncomfortable eyes. May be used with contact lenses.

Contraindications

No known contraindications.

Instructions for use

- Place 1 or 2 drops in affected eye as required
- Blink several times to wash the eye
- Use as often as required
- If you wear contact lenses there is no need to remove them

Precautions

- Do not use if the tamper-evident seal is broken or missing
- Do not swallow the solution
- Do not use if allergic to any of the ingredients
- To avoid contamination do not touch bottle tip on any surface and avoid contact with eyes
- Do not use after the expiry date
- Keep out of the reach of children
- Store between 5°C to 25°C
- Discard any remaining solution 45 days after first opening the bottle
- Potential risks, side effects or complications generally associated with eye drops are:
allergic reaction, inflammatory reaction or infections

If you have continued problems please discuss with your eye care professional.









Side Effects

No known side effects.

Contains:

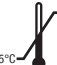Polyethylene glycol 400 (PEG 400) – 0.25%
OcuPure[®] preservative (sodium chlorite)
Sodium hyaluronate – 0.20%
Boric acid
Sodium borate (decahydrate)
Sodium chloride
Potassium chloride
Calcium chloride (dihydrate)
Magnesium chloride
Purified water

Symbols

 Consult instructions for use	 Manufacturer	 25°C 5°C Temperature limitation
 STERILE A Sterile. Aseptic technique	 LOT Batch code	 45D Discard any remaining solution 45 days after first opening the bottle
 Do not use if package is damaged	 Use by	



0483



AMO Ireland, Block B, Liffey Valley Office Campus
Quarryvale, Co Dublin, Ireland
© 2016 Abbott Medical Optics Inc.

 **Abbott**